


NE[®]

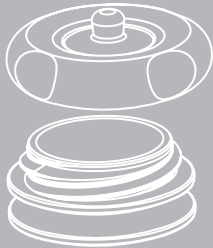

Code: NE032

NG GELTRODE

EEG only

The next generation
of the standard
EEG electrode

NE
neuroelectrics[®]

DESCRIPTION

The NG Geltrode consists of two pieces: the fastener (superior part) and the threaded washer (inferior part). The fastener is based on a Ag/AgCl sintered pellet with a 4 mm diameter.

MAINTENANCE


After each use, clean the NG Geltrode using a tissue and water. Avoid the direct sunlight exposure and the contact with metals when the electrode is not being used.

DURABILITY


The NG Geltrode is a long lasting electrode that can be reused many times. Its intended number of uses is 100.

USE INSTRUCTIONS


1 – Insert the threaded washer in the desired position of the neoprene cap, and put the cap on.

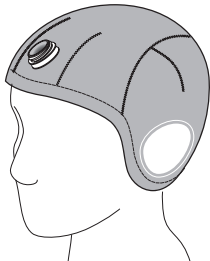

2 – Inject conductive gel inside the threaded washer with the syringe.

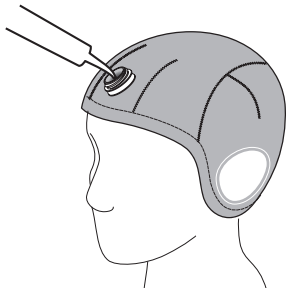

3 – Screw the fastener on the washer.

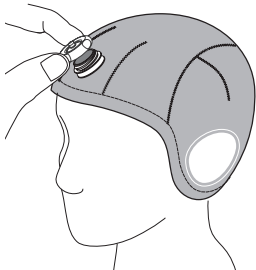

4 – Connect the cable to the electrode connector.

