

NE[®]



Code: NE025

STICKTRODE

Reference

Pre-gelled ECG, EOG,
CMS/DRL electrode



NEUROELECTRICS
BARCELONA
AV. Tibidabo 47 bis
08035 Barcelona
Spain



DESCRIPTION

The Sticktrode is a disposable pre-gelled electrode. It has an adhesive side with non-irritating gel and a 24 mm diameter.

When a pair of Sticktrodes is placed on the bare skin mastoid and connected to the CMS and DRL channels, it works as an electrical reference of the EEG system. Additionally, it can be used for ECG, EMG or EOG (when placed near the eye).

ORIGINAL MANUFACTURER REFERENCE


The original manufacturer is Covidien, and the code of the model is H124SG.

DURABILITY


The shelf life of the electrode is 24 months when stored at 10 - 30 °C temperature. The Sticktrode is disposable, it should only be used once.

USE INSTRUCTIONS


1 - Remove the adhesive cover.

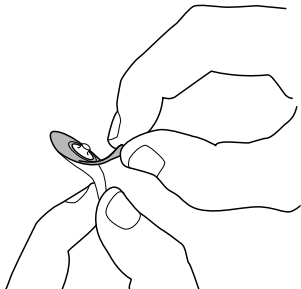

2 - Place the first electrode on the mastoid and ensure that there is no hair underneath.

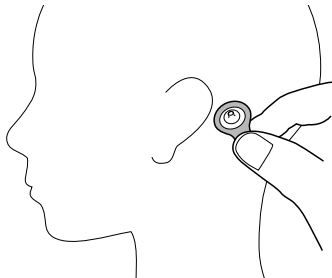

3 - Place the second electrode (DRL) inferiorly to the CMS electrode.

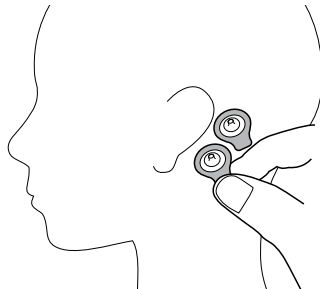

4 - Connect the CMS and DRL cable connections to the Stickrodes.

