


# NE<sup>®</sup>

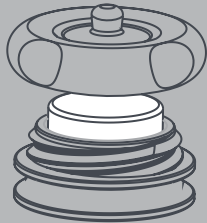

## SOLIDGELTRODE

EEG only

The solid gel  
EEG electrode



NEUROELECTRICS  
BARCELONA  
AV. Tibidabo 47 bis  
08035 Barcelona  
Spain

Code: NE028 & NE32



### DESCRIPTION

The NG Geltrode consists of two pieces: the fastener (top part) and the threaded washer (bottom part). The fastener is based on a Ag/AgCl sintered pellet with a 4 mm diameter.

### MAINTENANCE

After each use, clean the electrode using a tissue and water. Avoid direct sunlight exposure and contact with metals when the electrode is not being used. The units that are not used after the package is opened should be kept in a closed container to avoid air exposure.

### MAINTENANCE

After each use, clean the Solidegeltrode using a tissue and water. Avoid direct sunlight exposure and contact with metals when the electrode is not being used. The units that are not used after the package is opened should be kept in a closed container to avoid air exposure.

## DURABILITY


The Solidgeltrode is a long lasting electrode that can be reused many times. Its intended number of uses is 100.

The solidgel consumable can be used for a maximum of 24 hours. Since it is a disposable, it should be used only once.


## USE INSTRUCTIONS

- 1 – Insert the threaded washer in the desired position of the neoprene cap.
- 2 – Put the cap on.

3 –

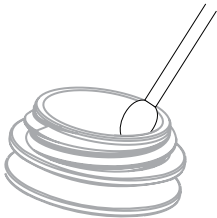

- 4 – Insert the solidgel consumable inside the threaded washer.

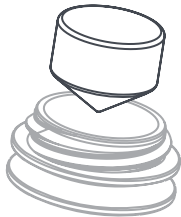

- 5 – Screw the fastener on the washer.



- 6 – After use, unscrew the fastener, then remove the solidgel consumable, and finally remove the headcap.