

NE®



## NG GELTRODE

EEG only

The next generation of  
the Standard electrode



NEUROELECTRICS  
BARCELONA  
AV. Tibidabo 47 bis  
08035 Barcelona  
Spain

Code: NE032



### DESCRIPTION

The NG Geltrode consists of two pieces: the fastener (top part) and the Threaded washer (bottom part). The fastener is based on a Ag/AgCl sintered pellet with a 4 mm diameter.

### MAINTENANCE

After each use, clean the NG Geltrode using a tissue and water. Avoid direct sunlight exposure and contact with metals when the electrode is not being used.

### DURABILITY

The NG Geltrode is a long lasting electrode that can be reused many times. Its intended number of uses is 100.


### Solidgel Consumable

It can be used for a maximum of 24 hours. Since it is a disposable, it should only be used once.


## WET USE INSTRUCTIONS

**1** – Insert the threaded washer in the desired position of the neoprene cap, and put the cap on.

**2** – Inject conductive gel inside the threaded washer with the syringe

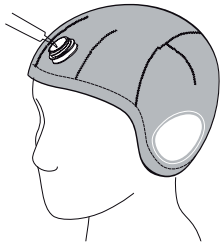

**3** – Screw the fastener on the washer and connect the cable to the electrode connector.

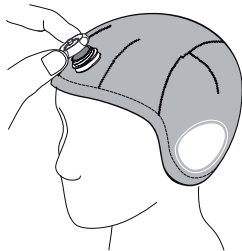

## DRY USE INSTRUCTIONS

**1** – Insert the threaded washer in the desired position of the neoprene cap, and put the cap on.

**2** – Using a cotton swab immersed in a water and glycerin solution (1:1), part the hair and clean the scalp with the swab through the washer.

**3** – Insert the solidgel consumable into the threaded washer and connect the cable to the electrode connector.

**4** – After use, unscrew the fastener, then remove the consumable.

